**IMPORTANTE**

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**SENZA VALVOLA****IMPORTANT**

Before use, check which class the mask belongs to according to EN149: 2001 + A1: 2009 and its suitability for the context of use. The class indication can be found on the box and on the half mask. The product usage data limit is on the packaging. The tightness of the filtering facepiece can be compromised if worn by people with beards.

**APPLICATIONS**

The disposable PPE Sicura P20 without valve/P20 with valve produced by Metalsud Lo Gatto srl guarantee the protection against chemical and biological risk from non-oily dust, fog, smoke, aerosols, bacteria, viruses and natural and artificial fibres. It is advisable to check the integrity of the mask before use. If you notice tears and/or stains, ask for a replacement mask. The wording NR in the marking indicates that the mask is not reusable and must be discarded at the end of the work shift (8 hours max). The protection offered by a Sicura P20 filter half mask without valve/P20 with valve includes that offered by class FFPI, the presence of the valve (where present) guarantees a better exhalation and therefore gaseous exchange with the external environment. The high efficiency filtering factor is guaranteed by the composition and interaction between the various woven layers that make up the body of the mask due to the filtering properties of the materials used: Warm Cotton, SS Spunbond and Meltblown. The harness of the head is equal to the date of manufacture and expiration of the complete device.

**INSTRUCTIONS FOR FITTING THE MASK**

- Insert a finger into the mask. Curve the nose bridge around the finger to give the shape of the nose.
- Holding the mask, put it on your face and hook the rubber bands behind your ears.
- Using both hands, shape the clip around your nose.
- Control test of the correct positioning of the mask with and without valve:
  - Place your hands on the mask as in figure 4 and inhale vigorously. If the air comes out of the edges of the mask or if it flows into the mask, from the edges, reposition the half-mask on your face.
  - (Only for mask with valve) Check the functioning of the exhalation valve and if it moves forward/backward following the course of the breath.
- Check the correct positioning of the mask with valve: Replace the mask immediately if you experience difficulty in breathing or if the mask is damaged, or if there are problems with adherence to the face. Careful observance of these instructions is essential for safe use of the respirator.
- Do not wash and/or make modifications.

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**WITH VALVE**

**STORAGE, CLEANING, MAINTENANCE, REVISION AND DISINFECTION INSTRUCTIONS**  
Store in a cool and dry place. Do not store at temperatures below 20 °C and above 30 °C. Do not expose to direct sunlight, UV light and fluorescent lamps. If the packaging is damaged or wet, discard the product. Do not use the disposable device beyond the expiration date indicated on the packaging.

**DISPOSAL METHODS**

Disposal must be carried out in unsorted collection containers. In case of use in healthcare, the mask must be disposed of in accordance with company procedures.

**CONTRAINDICATIONS**

The device can generate intolerances or individual hypersensitivity to the materials that compose it.

Loss of seal requirements are unlikely to be met if facial hair passes under the face seal; air quality (contaminants, oxygen deficiency); use of the equipment in an explosive atmosphere.

**UNI EN 149: 2001 + A1: 2009 class – PPE CAT. III**

<b>Model</b>	Sicura P20 without valve Sicura P20 with valve
<b>CE 2841</b>	Issued by Notified Body MNA LABORATUVARLARI SAN. TIC. LTD. ŞTI. Küçükbakkalköy, Yenidoğan Cd. No:21, 34750 Ataşehir/İstanbul, Turchia www.mnalab.com
<b>Use Limit</b>	12XTLV

Personal protective equipment compliant with EN149: 2001 + A1: 2009 and the general provisions for PPE referred to in EC Reg. 425/2016.

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